

Wall bars

The classic wall bars for private and therapeutically use. The frame is an untreated pinewood frame has oval birchwood bars. Inclusive assembly instruction and angle irons, plugs and screws for the installation on a cement wall. Delivery: unasssembled in carton. Loadable: 100 kg.

T3427 BxH 80x210 cm, 9 bars **pc**

T3428 BxH 80x230 cm, 10 bars **pc**

T3429 BxH 100x230 cm, 10 bars **pc**



Assembly instructions for wall bars

01. Lay one of the long frame-side bars on the floor, the holes for the bars have to be up to the front. (A)
02. Put in all the bars(B). This works easier if you use an hammer. (But put a piece of wood onto the bar before hitting it into the frame to prevent damaging it) Pay attention the bars with a drilling have to go together with the frame which has drillings in the hole.
03. Take the second frame-side bar (C) and put in the bars gradually (D). Let you help by a second person, that simplify the installation of the bars.
04. Now turn in the screws with the help of the key (E) into the bars. Just turn on the srews slightly. After all screws are turned in, you should turn them up more.
05. Now put up the wall bars upright and attach the fixing brackets (F) with the screws onto the bars
06. Make sure the wall bars stand vertically. Draw the dowel holes, drill and screw together the fixing brackets onto the wall.
07. Before use it make sure everthing is absolutely solid.

